**INSTRUCTIONS FOR USE**  
Important Information – Please Read Prior to Use

**Rx Only**  
**CE** 0086

 Orthofix Inc.  
3451 Plano Parkway  
Lewisville, Texas 75056-9453 U.S.A.  
1-214-937-3199  
1-888-298-5700  
www.orthofix.com

 Medical Device Safety Service  
(MDSS)  
Schiffgraben 41  
30175 Hannover  
Germany  
+49 511 6262 8630  
www.mdss.com

---

Device System Name:  
**SKYHAWK<sup>®</sup> Lateral Interbody Fusion System**

 Click directory below for desired language

<b>English</b> <b>EN</b>	<b>2-3</b>
<b>Español</b> <b>ES</b>	<b>4-5</b>

# INSTRUCTIONS FOR USE

Important Information – Please Read Prior to Use

Rx Only  
CE 0086

 Orthofix Inc.  
3451 Plano Parkway  
Lewisville, Texas 75056-9453 U.S.A.  
1-214-937-3199  
1-888-298-5700  
www.orthofix.com

 Medical Device Safety Service  
(MDSS)  
Schiffgraben 41  
30175 Hannover  
Germany  
+49 511 6262 8630  
www.mdss.com

## English EN

### Device System Name:

## SKYHAWK® Lateral Interbody Fusion System

### Description:

The SKYHAWK Lateral Interbody Fusion System consists of implants, trials, and instruments and is comprised of a variety of implants fabricated and manufactured from polyetheretherketone (PEEK) as described by ASTM F2026 with tantalum markers as described by ASTM F560. PEEK is utilized due to its radiolucent property, which aids the surgeon in determining if fusion in the operative site has occurred. Since PEEK is transparent in x-rays, tantalum marker pins are inserted into the implants in order to give surgeons a visual aid in determining the location of the implants, both intraoperatively and postoperatively.

The SKYHAWK Lateral Interbody Fusion System implants are offered in parallel and lordotic profiles to restore the natural curvature of the spine; the device may be implanted using a lateral or anterolateral approach.

The SKYHAWK Lateral Interbody Fusion System implants, trials and instruments are provided non-sterile. They require sterilization prior to use.

### Indications for Use:

When used as an intervertebral body fusion device, the SKYHAWK Lateral Interbody Fusion System is indicated for use with bone graft (autograft bone or allogenic bone graft composed of cancellous or corticocancellous bone graft) in patients with degenerative disk disease (DDD) at one or two contiguous levels from L2 to S1. These DDD patients may have up to Grade I Spondylolisthesis or retrolisthesis at the involved levels. DDD is defined as discogenic back pain with degeneration of the disc confirmed by history and radiographic studies. These patients should be skeletally mature and have had six months of non-operative treatment. Patients with previous non-fusion spinal surgery at the treated level may be treated.

The SKYHAWK Lateral Interbody Fusion System is intended to be used with supplemental fixation systems that are cleared by the FDA for use in the lumbar spine.

The SKYHAWK Lateral Interbody Fusion System must be used with autograft or allogenic bone graft composed of cancellous or corticocancellous bone graft.

### Contraindications:

The SKYHAWK Lateral Interbody Fusion System, as with other orthopedic implants, is contraindicated for use in patients with:

1. Active infections in which the use of an implant could preclude adequate and appropriate treatment of the infection.
2. Rapidly progressive joint disease or bone absorption syndromes such as Paget's disease, osteopenia, osteoporosis, or osteomyelitis which may prevent adequate fixation.
3. Conditions that may place excessive stresses on bone and implants, such as severe obesity, pregnancy or degenerative diseases. The decision to use this system in such conditions must be made by the physician taking into account the risks versus the benefits to the patient.
4. Known or suspected metal allergies.
5. Prior fusion at the level to be treated.

### Potential Adverse Events:

Potential adverse events include, but are not limited to:

1. Failure of the device to provide adequate mechanical stability.
2. Loss of fixation of the implant.
3. Device component failure.
4. Migration or bending of the device.
5. Loss of bony alignment.
6. Non-union.
7. Fracture of bony structures.
8. Resorption without incorporation of any bone graft utilized.
9. Immunogenic response to the implant materials.

**Note:** As with any major surgical procedure, there are risks involved in orthopedic surgery. Infrequent operative and postoperative complications known to occur are: early or late infection, which may result in the need for additional surgeries, damage to blood vessels, spinal cord or peripheral nerves, pulmonary emboli, loss of sensory and/or motor function, impotence, permanent pain and/or deformity. Rarely, some complications may be fatal.

### Warnings and Precautions:

The surgeon should be aware of the following when using implants:

1. The correct selection of the implant is extremely important. The potential for success is increased by the selection of the proper size, shape and design of the implant. The size and shape of human bones present limiting restrictions on the size and strength of implants. No implant can be expected to withstand the unsupported stresses of full weight bearing.
2. Correct handling of the implant is extremely important. Implants should not be bent, notched or scratched. These operations can produce defects in surface finish and internal stress concentrations, which may become the focal point for eventual failure of the device.
3. DO NOT USE DAMAGED IMPLANTS.
4. Single Use Only – SKYHAWK Lateral Interbody Fusion System implants are SINGLE USE ONLY. Surgical implants should not be re-used. Any implant once used should be discarded. Even though the device appears undamaged, it may already have small defects and internal stress patterns that may lead to fatigue failure. Re-use could result in injury or require re-operation due to breakage or infection.
5. Non-sterile; the SKYHAWK Lateral Interbody Fusion System implants and instruments are provided non-sterile, and therefore must be thoroughly cleaned and sterilized before each use.
6. Postoperative care is important. The patient should be instructed in the limitations of the implant and should be cautioned regarding weight bearing and body stress on the device prior to secure bone healing.
7. Patients with previous surgery at the levels to be treated may have different clinical outcomes when compared to those without a previous surgery.
8. The implantation of the intervertebral body fusion device should be performed only by experienced spinal surgeons with specific training in the use of this device because it is a technically demanding procedure presenting a risk of serious injury to the patient.

### MRI Compatibility Information:

The SKYHAWK Lateral Interbody Fusion System has not been evaluated for safety and compatibility in the Magnetic Resonance (MR) environment. The system has not been tested for heating or migration in the MR environment.

### Cleaning:

Implants are provided clean but not sterile. Once an implant comes in contact with any human tissue or bodily fluid it should not be re-sterilized or used. Please discard all contaminated implants.

All instruments must be thoroughly cleaned prior to use and after each re-use of the instruments. Reprocess all instruments as soon as it is reasonably practical following use to prevent drying. Cleaning may be done using validated hospital methods or following the validated cleaning processes described below.

None of the instruments require disassembly prior to cleaning.

### From Point of Use:

Wherever possible, do not allow blood, debris or body fluids to dry on instruments. For best results and to prolong the life of the surgical instrument, reprocess immediately after use.

### Preparation for Cleaning:

1. Any instruments with moving parts (e.g., knobs, triggers and hinges) should be placed in the open position to allow better access of the cleaning fluid to the difficult to clean areas.
2. Soak the instruments for a minimum of 10 minutes in sterile water prior to the manual or automated cleaning process.
3. Use a soft cloth or plastic bristle brush to remove any visible soil from the instruments prior to manual or automated cleaning.

### Manual Cleaning:

1. Because none of the instruments can be disassembled, use a soft cloth and/or a soft plastic bristle brush to remove any visible soil from the outside and inside of the instrument.
2. Prepare Vesphene® Ilse agent at the use-dilution recommended by label direction – 1 ounce per gallon or 1 mL of Vesphene® Ilse to each 128 mL of potable tap water per manufacturer's recommendations.
3. Bathe instruments in prepared room temperature solution as recommended by the detergent manufacturer.
4. Manually agitate instruments in Vesphene® Ilse solution for 15 minutes.
5. Scrub instruments with a soft plastic bristle brush if visible soil is noted and use a soft plastic bristle brush or pipe cleaner long enough to reach the entire length of the interior lumen to remove the soil. Rinse the instruments in USP <1231> purified water for 1.5 minutes.
6. Hang dry the device.
7. Visually inspect the instruments for visible soil.
8. If visible soil is noted, repeat the steps listed above.

**Automated Cleaning:**

1. Use a soft cloth or soft plastic bristle brush to remove any visible soil from the outside and inside of the instruments. Use a soft plastic bristle brush or pipe cleaner long enough to reach the entire length of the interior lumen to remove the soil from the inner lumens prior to automated cleaning.
2. Set up the washer/disinfector detergent dose as ½ ounce of Endozime AW Plus® per gallon of water or according to the manufacturer’s recommendations.
3. Remove the instruments from the instrument case and place them into the washer baskets.
4. Orient instruments in the automated washer’s carriers as recommended by the washer manufacturer.
5. The following automated cleaning cycle is recommended (minimum recommended times are provided for each stage):
  - a. Pre-Wash 1: cold potable water, 2 minutes
  - b. Enzyme/Detergent treatment:
    1. Spray, 20 seconds
    2. Soak, 1 minute
    3. Rinse cold potable water, 15 seconds
    4. Rinse cold potable water, 15 seconds
  - c. Wash ≥ 65°C, 2 minutes using Endozime AW Plus®
  - d. Rinse 1: hot potable water, 15 seconds
  - e. Rinse 2: hot potable water, 15 seconds
  - f. Rinse 3: hot potable water, 15 seconds
  - g. Rinse 4: hot potable water, 15 seconds
  - h. Thermal rinse ≥ 93°C, 1 minute
  - i. Heated USP <1231> Purified Water Rinse 1: re-circulating 10 seconds
  - j. Heated USP <1231> Purified Water Rinse 2: non re-circulating 10 seconds
  - k. Dry at 115°C, 7 minutes
6. Visually inspect the instruments for visible soil
7. If visible soil is noted, repeat the above listed steps until no visible soil is noted.

**Note:** Certain cleaning solutions such as those containing caustic soda, formalin, glutaraldehyde, bleach, and/or other alkaline cleaners may damage instruments. These solutions should not be used.

**Note:** Visually inspect instruments after cleaning and prior to each use. Discard or return to Orthofix if instruments are broken, discolored, corroded, have cracked components, pits, gouges, or otherwise found defective. Do not use defective instruments.

**Sterilization:**

SKYHAWK Lateral Interbody Fusion System implants and instruments are supplied NON-STERILE. Prior to use, all implants and instruments should be placed in the instrument/implant case which will be either wrapped in an FDA cleared sterilization wrap or placed in a rigid sterilization container and placed in the autoclave for sterilization by the hospital using one of the following recommended cycles:

**Sterilization in Orthofix Cases with Blue Wrap:**

- |                                |     |                                |
|--------------------------------|-----|--------------------------------|
| Method: Steam                  | or: | Method: Steam                  |
| Cycle: Gravity                 |     | Cycle: Prevac                  |
| Temperature: 270°F (132°C)     |     | Temperature: 270°F (132°C)     |
| Exposure time: 15 minutes      |     | Exposure time: 4 minutes       |
| Drying time: 30 minutes        |     | Drying time: 30 minutes        |
| Double wrapped                 |     | Double wrapped                 |
| (FDA cleared wrap recommended) |     | (FDA cleared wrap recommended) |

**Sterilization in Rigid Sterilization Containers:**

When using rigid sterilization containers, clean, inspect and prepare the rigid sterilization container according to the manufacturer’s instructions.

Select the appropriate rigid sterilization container with either a filtered or solid bottom to properly enclose the Orthofix case (recommended 23¼” long x 11¼” wide container). The following sterilization cycle has been validated:

- Method: Steam
- Cycle: Prevac
- Temperature: 270°F (132°C)
- Exposure time: 4 minutes
- Drying time: 30 minutes

**Note:** Rigid sterilization containers with solid bottoms cannot be used in gravity steam cycles.

Validation and routine monitoring should be performed per ANSI/AAMI ST79 *Comprehensive guide to steam sterilization and sterility assurance in health care facilities*. Other cycles may be used as long as they comply with the above practices and provide a sterility assurance level of 10<sup>-6</sup>.

**Packaging:**

Packages for each of the components should be intact upon receipt. If a consignment system is used, all sets should be carefully checked for completeness and all components should be carefully checked for damage prior to use. Damaged packages or products should not be used and should be returned to Orthofix.

The SKYHAWK Lateral Interbody Fusion System instruments and implants are provided in modular cases specifically intended to contain and organize the system’s components. The system’s instruments are organized into trays within each modular case for easy retrieval during surgery. These trays also provide protection to the system components during shipping. Additionally, individual instruments and implants are provided in sealed poly bags with individual product labels.

**Product Complaints:**

Any Health Care Professional (e.g., customer or user of this system of products) who has any complaints or who has experienced any dissatisfaction with the product quality, identity, durability, reliability, safety, effectiveness, and/or performance, should notify Orthofix Inc., 3451 Plano Parkway, Lewisville, TX 75056, USA, by telephone at 1-214-937-3199 or 1-888-298-5700 or by e-mail at [complaints@orthofix.com](mailto:complaints@orthofix.com).

**Further information:**

A recommended operative technique for the use of this system is available upon request from Orthofix at the numbers provided above.

**Latex Information:**

The implants, instruments and/or packaging material for the SKYHAWK Lateral Interbody Fusion System are not formulated with and do not contain natural rubber. The term “natural rubber” includes natural rubber latex, dry natural rubber, and synthetic latex or synthetic rubber that contains natural rubber in its formulation.

**Caution:** Federal law (USA) restricts this device to sale by or on the order of a physician.

<b>Rx Only</b>		Federal (U.S.A.) law restricts this device to sale by or on the order of a physician	
	See Instructions for Use		Catalogue Number
	<a href="http://Orthofix.com/IFU">Orthofix.com/IFU</a>		Manufacturer
	Single Use Only Do Not Reuse		Authorized Representative
	Provided Non-Sterile		Lot Number

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